




Esma's Exclusive, Low-Cost Alternative to Cassettes


(IB) Insertion basket system designed for Esma ultrasonic washers.

Full Cycle, Hands Free

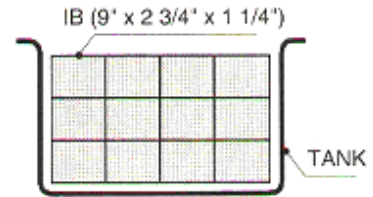

OPERATORY

- Break wrapping, or open bag.
- Place instruments on a tray.
- Return dirty instruments to the IB.

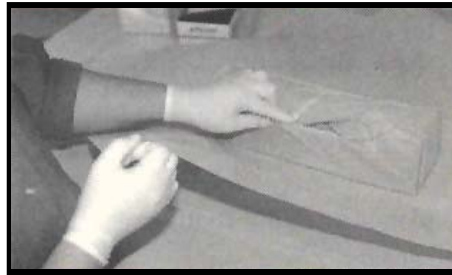


NO-SCRUB CLEANING

Place IB with dirty instruments into main basket of unit. Wait till main basket fills up. * Push button for cleaning action.


Remove when cycle is complete.

* You can start cleaning with less than (12) IB's in tank.

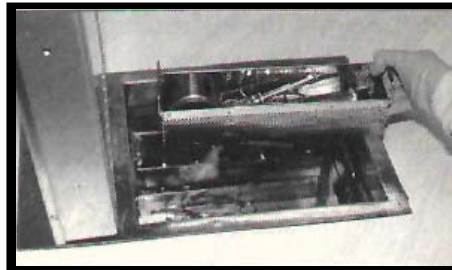


STERILIZATION

- Wrap baskets with clean instruments
- Sterilize wrapped baskets
- Remove for storage/operatory

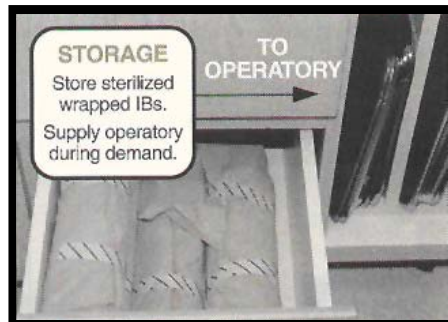

Safe, outstanding money saver. Pays for itself in 6-10 months.

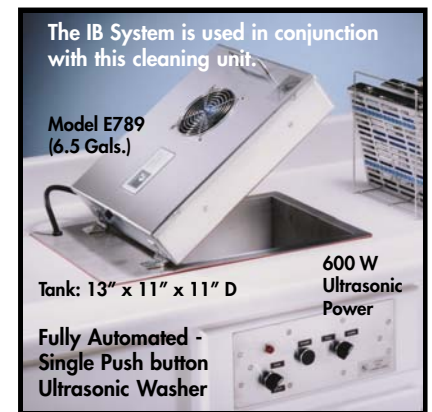


P.O. Box 734
450 Taft Drive
South Holland, IL 60473
800-276-2466
www.esmainc.com
Email: sales@esmainc.com

Insertion Basket Systems

Once in the IB, dirty instruments are not being touched. The IB completes a full-cycle and returns sterilized to the operator. A streamlined, safe, time-saving operation.



IB (9" x 2 3/4" x 1 1/4")

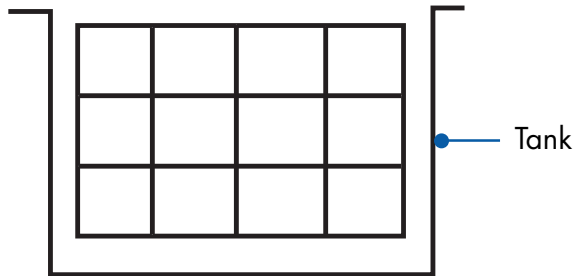

After accumulating twelve or less IBs in the main basket, the cleaning unit can be started by a single push button. No monitoring or timers or other work is required. The buzzer will sound at the end of cleaning.

STORAGE

- Store sterilized wrapped IBs.
- Supply operator during demand.

STERILIZATION

- Place IBs with clean dry instruments* into sterilizer.
- Sterilize for the proper durations.
- Remove from sterilizer. Wrap* or bag. The IB can be wrapped prior or after sterilization.

OPERATORY

- Break wrapping or open bag. Place instruments on a tray.
- Return dirty instruments to the IB

NO-SCRUB CLEANING

- Place IB with dirty instruments into main basket or unit. Wait until main basket fills up.
- Push button for cleaning action.
- Remove when cycle is complete.

*You can start cleaning with less than (12) IB's in tank

The only "hot spot" is the return of used instruments to the IB.

However, there is no exposure to sharp ends: instruments are being held by the middle of the shaft.

Stackable Insertion Baskets (IB)



In a group practice, each Doctor and Hygienist can have personalized IBs (initials punched in).

Here is how Dr. R in Colorado is using our unit:

They accumulate dirty instruments in the main basket and start the unit when leaving for lunch. Upon return, dry, clean instruments are ready for sterilization. The afternoon accumulation is cleaned by pushing the button just before leaving the office. Clean instruments are found next morning.



P.O. Box 734
450 Taft Drive
South Holland, IL 60473
800-276-2466
www.esmainc.com
Email: sales@esmainc.com