



Auxie Electropolishers

Auxie polishers eliminate most manual rubber wheeling.

The Electropolisher Series



How Does It Work

Load the frame onto the holding assembly and insert into the bath. Set voltage to 8 volts and turn timer to 30 minutes for upper frames or 20 minutes for lower frames.

Superior shine obtained only by using Esma-Brite and Auxie machine combination



P.O. Box 734
450 Taft Drive
South Holland, IL 60473
800-276-2466

Features & Specs



Model# E678-1
"Auxie-1"

For nearly 30 years Auxie Electropolishers have been the *workhorse* of the partial department. As a long-term investment with a short-term payback, Auxie units consistently elicit comments such as:

Maryland lab owner-*"Its like having another person. In the time it took to finish 5 eccases I now do 11."*

Ontario Lab-*"Best investment I made in years."*



Model# E678-3
"Auxie-3"

The "Auxies" when combined with ESMA-BRITE electropolishing solution result in superior shine and maximum labor savings.

- ◆ Unprecedented luster
- ◆ 50% of the current is directed inside the deep palate. *Never a "burned" clasp*
- ◆ Eliminate 80% of manual polishing



Model# E678-5
"Auxie-5"

Built to last Auxie units feature:

- ◆ Stainless steel cabinets
- ◆ Non-corroding electrodes
- ◆ Circuit breaker protection-No damage to partial if shorting occurs
- ◆ Independent operation of each station
- ◆ Temperature controlled heating and fan cooling-Increases life of polishing solution
- ◆ 1 year unconditional guarantee

Model:	Size:	Work stations:	Daily capacity:
E678-1	17"x11"x10"	1	1-10 partials
E678-3	14"x18"x17"	3	10-20 partials
E678-5	20"x18"x17"	5	20-40+ partials